

SIGVARIS



ULCER X – THERAPY SOLUTION



ULCER X


HEAL YOUR PATIENTS' ULCERS FASTER

Heal Your Patients' Ulcers Faster

When healing a venous leg ulcer, the treatment choice has traditionally been firm, non-elastic bandages. However, a recent study conducted in Italy at the University of Siena and the University of Ferrara has determined a more effective treatment:

The SIGVARIS ULCER X Kit.

As part of the SIGVARIS Medical Therapy collection, the ULCER X Kit provides a therapeutic solution with a 96.2% healing rate for venous ulcers (versus 70% for bandages). With half the healing time as the bandage alternative for ulcers up to 4cm in diameter* (see Table 1) and noticeably less pain and discomfort than the traditional solution, the SIGVARIS ULCER X Kit is the ideal answer to your patients' needs.

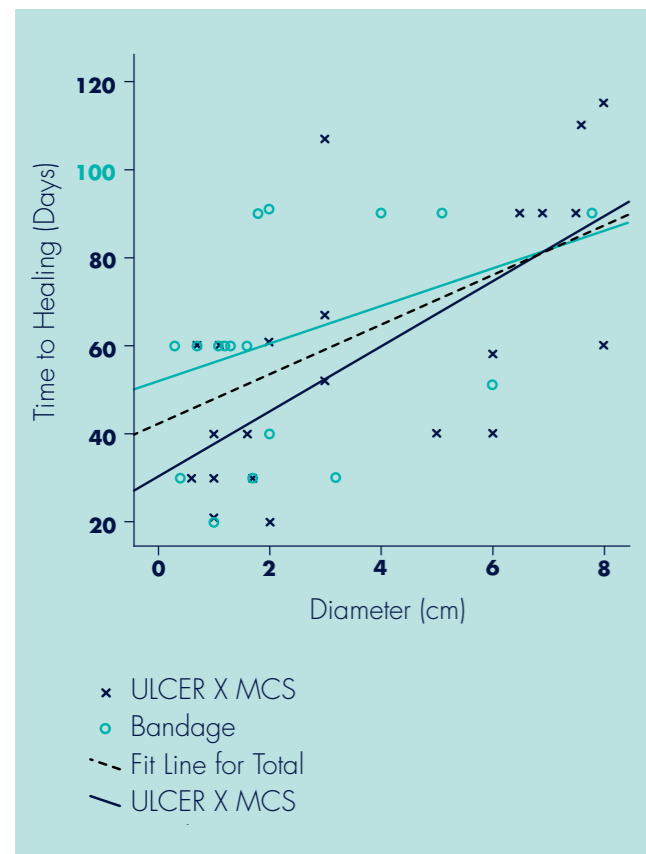

Study Methodology:

- Multi-center randomized trial with 60 patients
- Treated with either short stretch multi-layer bandages or a two-stocking system (SIGVARIS ULCER X Kit)
- Patient characteristics and ulcer features were evenly distributed
- Proportion of ulcers healed within 4 months and the time to completion of healing were recorded
- Subjective appraisal was assessed with validated questionnaire

* Healing time was equal to bandages for ulcers greater than 4cm



The ULCER X Kit

The ULCER X Kit is made of two components: a low-compression understocking (~15 mmHg) and a 23–32 mmHg overstocking, providing a combined ankle pressure of ~40 mmHg. The understocking is made with an inner layer of cotton and a smooth exterior that allows easy donning of the overstocking.

The 23–32 mmHg overstocking is recommended for removal at night, but your patient should wear the low-compression understocking during sleep or while recumbent.

«Discomfort during the night and in the morning was absent with the ULCER X Kit.»

The treatment of venous leg ulcers with a specially designed compression stocking kit. F. Mariani, et al. Phlebologie 2008; 37: 191–197. September 2008.


The SIGVARIS ULCER X makes it easy for your patients to comply with venous leg ulcer therapy treatment.

ULCER X

WHY PRESCRIBE

SIGVARIS

Prevalence of pain, discomfort and hindrance of activities (percentage of patients with at least one point on each Likert scale) ‡

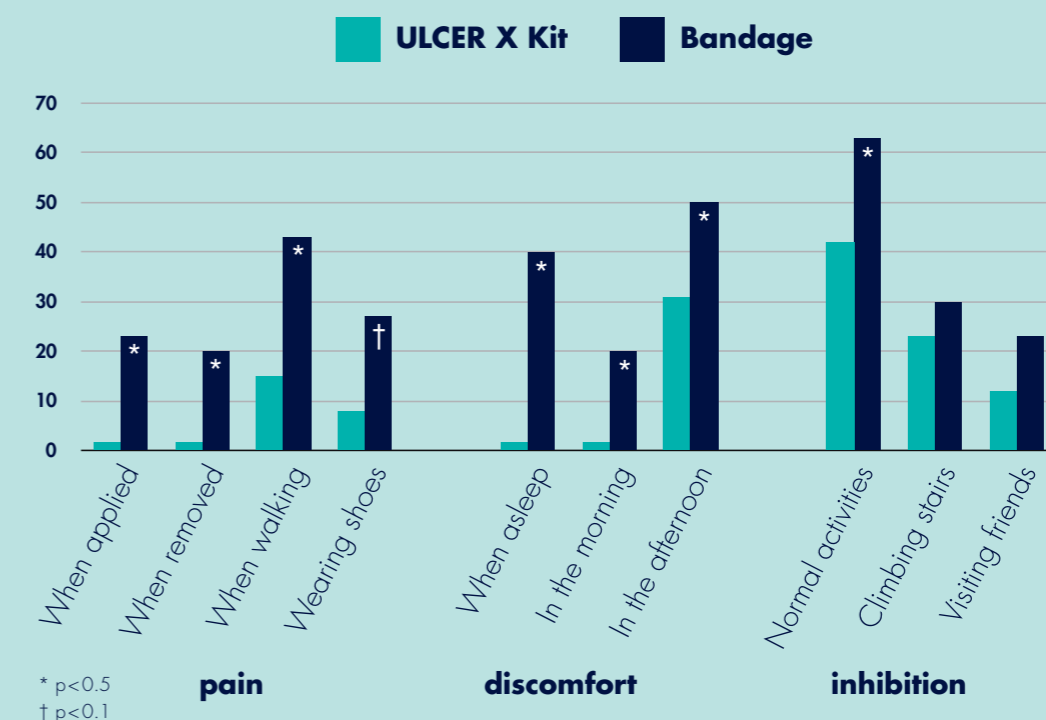

‡ Source: The treatment of venous leg ulcers with a specially designed compression stocking kit. F. Mariani, et al. Phlebologie 2008; 37: 191–197. September 2008.

Why Prescribe

Scientifically proven results:


- Complete wound closure achievement: 96.2% (25/26) with ULCER X Kit (p=0.011) as compared to 70% (21 of 30) with bandages
- Ulcers with diameter of up to approximately 4cm healed twice as fast with the ULCER X Kit compared to bandages *Larger ulcers (>4cm) healed as rapidly with the ULCER X Kit as with bandages*
- Reported pain at night/morning was absent with the ULCER X Kit group but reported by 40%/20%, respectively, in the bandage group

ULCER X Benefits:

- Ease of application promotes compliance as compared to bandaging, which requires medical expertise for application
- Successful treatment without compromising wound dressing
- Significantly-reduced pain lets your patient rest comfortably

ULCER X

MEASUREMENTS & ASSORTMENT

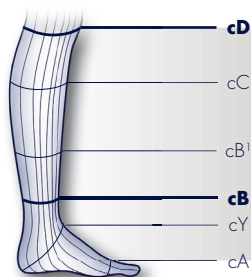

Circumference in cm	small	small PLUS	medium	medium PLUS	large	large PLUS
cD	28-34	32-38	31-37	36-42	34-40	40-46
cC	31-36	35-40	34-39	39-44	37-42	43-48
cB ¹	24-29	24-29	29-32	29-32	32-36	32-36
cB	19-23	19-23	23-26	23-26	26-29	26-29
cY	28-32	28-32	32-35	32-35	35-38	35-38
cA	19-23	19-23	23-26	23-26	26-29	26-29



Length in cm	
short	33-37
long	38-44

Material composition:

TRADITIONAL CALF:
70% polyamide, 30% elastodiene

UNDERSTOCKINGS:
62% polyamide, 26% elasthan,
12% cotton

- Wash daily using a laundry bag
- Do not use fabric softener
- Do not place in dryer
- Do not iron
- Do not dry clean

c = circumference
ℓ = length



ULCER X KIT	
TRADITIONAL CALF	UNDERSTOCKING
1 piece A-D Calf Compression class 2 Open toe Colour beige Sizes: small, small PLUS, medium, medium PLUS, large, large PLUS	2 pieces Understockings Closed toe Colour beige Sizes: small, medium, large adapted to the required size of the TRADITIONAL calf

Understockings are available separately – packaging consists of 4 understockings.

Colour
BEIGE

SIGVARIS AG
Gröblistrasse 8
CH-9014 St. Gallen
Tel. +41 (0)71 279 33 66
Fax +41 (0)71 274 29 29
www.sigvaris-export.com

SIGVARIS is the world market leader in the field of medical compression stockings, available in over 70 countries.

SIGVARIS, LIFE FOR LEGS and the leg icon are registered trademarks of Sigvaris AG, CH-9014 St.Gallen/Switzerland, in many countries worldwide.

© 2011 Copyright by Sigvaris AG, St. Gallen/Switzerland

